



Material Helpi® Ring:
100 % Thermoplastic Elastomer (TPE)

Material blue sleeve:
Polycarbonate (PC)

EAN 4013273000574

Manufacturer:
KESSEL medintim GmbH
Nordendstr. 82 – 84
64546 Mörfelden-Walldorf
Germany
E-Mail: export@medintim.de

www.medintim.de

Helpi® Ring

For men who come
(too) early.

What is the Helpi® Ring?

The Helpi® Ring keeps your erections hard and prevents premature, visible ejaculation.*
A simple solution without drugs, creams or gels.

The Helpi® Ring is a safe and easy way of removing stress.

*Note:

Premature ejaculation (ejaculatio praecox) may be a symptom of a condition requiring treatment, such as prostatitis or hyperthyroidism. So it's important to consult a doctor as soon as possible.

Advantages:

- friendly to the skin, soft, flexible
- reusable
- compatible with water-soluble lubricants, silicone oil, skin creams, vegetable fats
- biocompatible, tested to ISO 10993
- compatible with condoms made from latex and polyisoprene
- free from latex, phthalate, bisphenol A and silicone

How to use:

Stretch the ring onto the blue sleeve (Fig. 1). Push the sleeve, together with the ring, over the penis towards your body (Fig. 2). Slip the ring off the sleeve and onto the penis (Fig. 3). Now remove the sleeve (Fig. 4). The Helpi® Ring will now maintain the erection regardless of when the user comes. The erection is maintained even after the man climaxes. He can continue to have sex. The Helpi® Ring does not interfere and can even have a stimulating effect on the partner (Fig. 5). The user can easily remove the Helpi® Ring at any time. Pull on the flaps and wait a few seconds until the erection subsides. You can now simply slip the ring off (Fig. 6).

