

Note:

We recommend trying out Waypi® under the shower at home, before taking it on the road. The use of a urinating funnel may be unfamiliar at first. The Waypi® urinating funnel is available in pharmacies or from specialist retailers.



New!

Waypi®

Urinating funnel for travel
and other situations

Material: 100 % TPE, odorless



EAN 4013273002127

Made in Germany

Manufacturer:

KESSEL medintim GmbH
Nordendstr. 82 – 84
64546 Mörfelden-Walldorf
Germany

Phone: +49 6105 20 37 20

www.medintim.de

05.03.2021

Medintim

Whether out in nature, while hiking, at festivals or at the service station: non-existent or dirty toilets are no longer a problem with the Waypi®. Urinate standing upright - hygienic, simple and with a handy storage box.

Features of the Waypi®:

- extra smooth, satin finished surface
- easy to clean
- conforms to body contours
- innovative shape prevents overflow and splashing
- with a loop for practical folding
- with matching storage box
- also available with drain hose on request
- fits all bottles

How to use the Waypi® correctly:

reusable,
ecological,
no unnecessary
waste

Take the still folded Waypi® out of its storage box. To unfold, simply lift the loop and the Waypi® unfolds by itself.

To use, hold the Waypi® urine funnel with the loop towards the body. The end of the funnel points diagonally forward towards the ground. Make sure that the Waypi® is covering the urethral outlet.

After use, clean the Waypi® with soap and water, fold it and store it in the storage box. It has a small ventilation slot on the underside to avoid moisture collecting in the box.

